**Lapisa**<sup>®</sup>  
SALUD ANIMAL

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## Lapisa Implanter Instructions for Use and Care

### Product Highlights

- Ergonomic Design
- Produced from durable materials to ensure long service life
- Spare needle compartment for fast field changeover
- Pusher bar feature for proper implant placement


### Step 1: Remove gun from packaging

### Step 2: Attaching the needle


- Remove the needle nut from the front of the implanter
- Insert the base of the needle onto the implanter
- Slide needle nut over needle and tighten by rotating clockwise
- NOTE: The Lapisa Implanter is equipped with a needle drawer containing a replacement needle. The drawer is easily accessed at the top of the gun and allows the user to safely keep a replacement needle on hand during field use.

### Step 3: Inserting the clip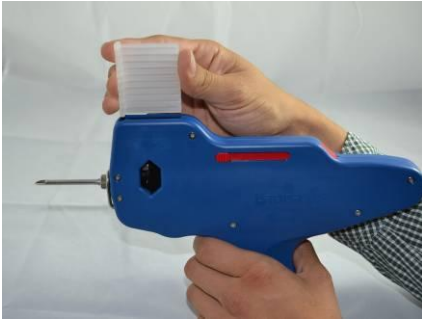

- Insert a pre-loaded 10 dose clip of **Maxichoice 200**® or **Maxibeef**® into the opening slot in the top of the implanter and press down until you feel it “click” into position.
- The gun is now loaded. **DO NOT PULL THE TRIGGER UNTIL READY TO DELIVER IMPLANT**

### Step 4: Delivering Implants

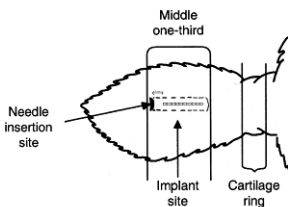- Prepare the ear for implanting by cleaning and disinfecting according to implant manufacturers instructions
- Grasp the ear and insert the needle between the skin and cartilage. Insertion is intended for the middle third of the ear
- Pull the trigger of the gun to administer implant. The gun will be pushed back by the “pusher bar” laying the implants in the intended position and not allowing them to bunch up in the injection site.

- Repeat steps until a new clip is needed. The clip will automatically advance with each new animal. Remaining doses can be seen through the window exposing the clip.
- After all doses are administered the clip is simply pulled out of the bottom of the gun for disposal

### Step 5: Cleaning and Maintenance

- The Lapisa Implanter should be wiped clean after use with a damp cloth. It is not recommended to immerse the implanter in water or cleaning solution.
- Needles are not intended to be sharpened in the field and should be replaced as needed.
- Store your Lapisa Implanter in a clean and dry environment to ensure a long reliable service life.

**MaxiBeef**<sup>®</sup>

**MaxiChoice 200**<sup>®</sup>

**Son marcas registradas de Lapisa<sup>®</sup>**

© copyright · Lapisa, S.A. DE C.V. · 10/14 · Reservado a los derechos de modificaciones técnicas y diseño · Printed in Germany  
© copyright · Lapisa S.A. DE C.V. · 10/14 · Subject to amendments in technology and design · Printed in Germany